

Cem-Ostetic™

CE
0123

Putty, Granules and Blocks



- Easy-to-use - Quick to Prepare (5 Minutes)
- 100% Synthetic (Calcium Salts Scaffold, Including Nanocrystalline Hydroxyapatite)
- 100% Resorbable and Effective (Three to Four Months in Animal Tests and Clinical Studies)
- Osteoconductive, Biocompatible and Neutral pH
- Safe, Sterilized by Gamma Irradiation (25 kGy Minimum Dose)

The putty formulation hardens in 5 minutes even in a wet or bloody environment.

To prepare the putty:


1. Quickly mix powder with sterile water provided to form homogeneous paste.
2. Shape implant manually or inject into void with syringe.
3. Let putty harden for 3 minutes — the implant is now ready.

	Catalog No.	Volume	Description
	PUTTY		
	CemO-02P	2.5 cc	Putty Kit (Sterile Water and Powder)
	CemO-05P	5 cc	Putty Kit (Sterile Water and Powder)
	CemO-10P	10 cc	Putty Kit (Sterile Water and Powder)
	CemO-20P	20 cc	Putty Kit (Sterile Water and Powder)
	CemO-30P	30 cc	Putty Kit (Sterile Water and Powder)
	GRANULES		
	CemO-02G	2.5 cc	Cancellous Granules; 0.5 - 1 mm Particle Size
	CemO-05G	5 cc	Cancellous Granules; 1 - 3 mm Particle Size
	CemO-10G	10 cc	Cancellous Granules; 1 - 3 mm Particle Size
	CemO-15G	15 cc	Cancellous Granules; 1 - 6 mm Particle Size
	CemO-30G	30 cc	Cancellous Granules; 1 - 6 mm Particle Size
	CemO-60G	60 cc	Cancellous Granules; 1 - 6 mm Particle Size

To order blocks, contact Berkeley Advanced Biomaterials technical support directly (see our address below).

Common use of Cem-Ostetic™ includes defect filling in total hip revision, spinal fusion, cranio-facial surgery, cranial vault remodeling, cranioplasty, hand or foot surgery, fracture repair, joint reconstruction, cyst treatment and limb salvage.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician or hospital. Cem-Ostetic™ is a trademark of Berkeley Advanced Biomaterials, Inc. Manufactured and distributed by Berkeley Advanced Biomaterials, Inc. San Leandro, CA, USA.

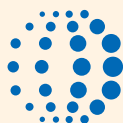

**Berkeley Advanced
Biomaterials, Inc.**

To order products from your local distributor, please contact Berkeley Advanced Biomaterials, Inc.
1933 Davis Street, Suite 307 - San Leandro, CA, U.S.A. Telephone: 510.883.1644 - Fax: 510.883.1315
E-mail: info@ostetic.com - Internet: www.ostetic.com or www.hydroxyapatite.com